WIRE PROGRAM

Dentaurum supplies wires and arches of different alloys under the following names:

Product

1. Remanium®
2. Remaloy®, Redur, Crozat
3. Remattan®
4. Remattan® "LITE"
5. Dentaflex®

Alloy Composition

- Stainless Steel
- Cobalt Base
- Nickel-Titanium
- Nickel-Titanium
- Stainless Steel

Major Components

- FeCr18Ni9
- CoNi22Cr18Mo4W4Fe5
- NiTi45
- NiTi45
- FeCr18Ni9

Remaloy®, Redur and Crozat wires have the same properties. Redur serves as class wire in prosthetics. Crozat wire is used for Crozat appliances in jaw orthopedics. All other wires are used in orthodontic treatment.

Figure 1 illustrates the flexural characteristics of Dentaurum's wires. The bending curves are adjusted for direct comparison with spring-tempered steel and are without regard to the wire dimensions. Table I shows suggested applications of Dentaurum wires.

Dentaflex® six-strand (No. 5 in Figure 1)

Highly flexible, therefore, suitable for initial treatment, particularly for leveling.

Dentaflex® triple-strand (No. 6 in Figure 1)

Provides slightly higher forces than Dentaurum six-strand. Also used for leveling in the initial phase of treatment.

Dentaflex® eight-strand (No. 7 in Figure 1)

Especially suitable for torquing during the initial treatment. Also applicable as a finishing wire.

Remattan® "LITE" (No. 4 in Figure 1)

An alternative to dentaflex®. Good flexibility accompanied by high resiliency at almost constant force conditions. The high activation capability results in longer treatment intervals. Little friction.

Remattan® (No. 4 in Figure 1)

Similar to remattan® "Lite", but with a somewhat higher force level. It is suitable for the initial treatment through the intermediate stage. Resiliency is good and comparable to remattan® "Lite". Not suitable for loops.

Remaloy®, heat treated (No. 1 in Figure 1), not heat treated (No. 3 in Figure 1)

An attractive alternative to remanium® if the treatment involves loops at a high force level because it is bendable in its non-tempered state. Its application is in the intermediate and finishing phase of treatment.

Remanium®, spring hard (No. 2 in Figure 1)

Still the best solution if higher forces are indicated. Incremental strength. Good formability and weldability. May be used from the initial phase through to the finishing phase.

Fig. 1

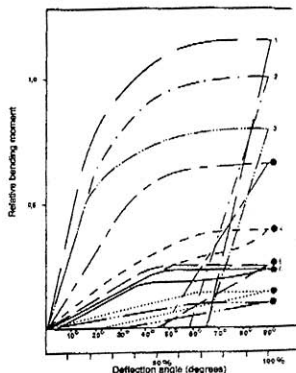
Table I

Suggested Applications for Dentaurum Wires and Arches	Treatment Stages										
	Initial Phase			Intermediate Phase				Finishing Phase			
	Tooth Leveling	Torque	Rotation	Tooth Bow Equalization	Torque	Rotation	Retraction	Tooth Bow Equalization	Torque	Rotation	Fixing
Dentaflex® 8 x square/rectangular	●	●	●								●
Dentaflex® 6 x round	●		●								
Dentaflex® 3 x round square/rectangular	●	●	●								
Remattan® "LITE" round square/rectangular	●	●	●								●
Remattan® round square/rectangular	●	●	●		●	●					●
Remaloy® round square/rectangular	●			●	●	●	●	●	●	●	●
Remanium® round square/rectangular	●		●	●	●	●	●	●	●	●	●

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